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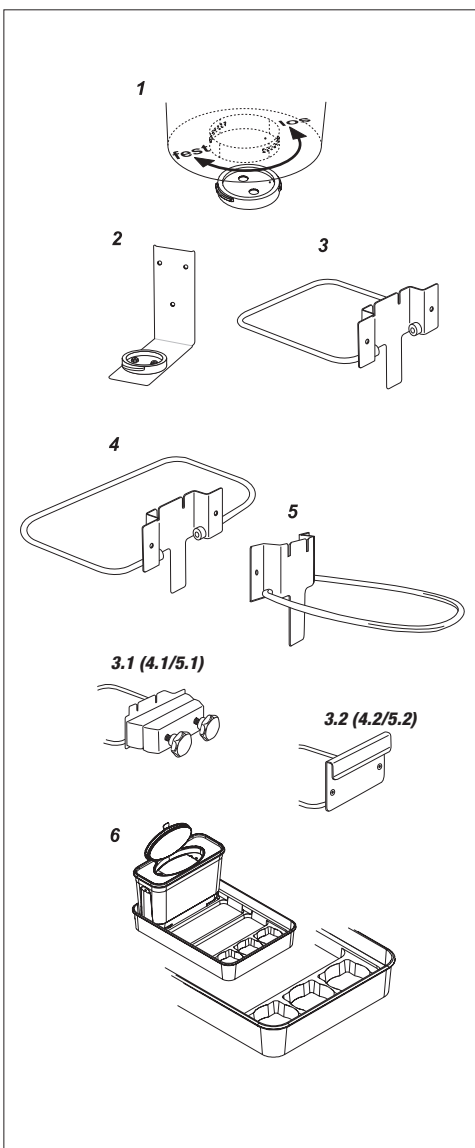
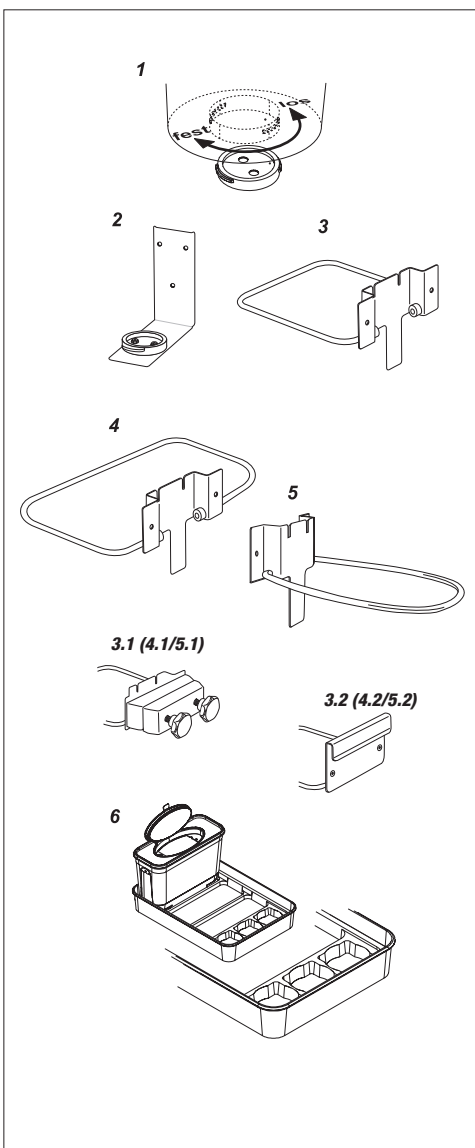
EN Instructions for Use: Holding Fixtures for Multi-Safe Disposal Boxes

1. Base adapter (REF 95.965.001)

Turn the box upside down. Place the adapter (1) into the recess in the box base and turn clockwise. Choose a place to position the box as well as the direction in which the stripping/removal aids are to point. Ensure that the surface where the base adapter (1) is to be attached is free from dust and grease. Remove the protective paper from the adapter, place the box onto the selected flat surface complete with the base adapter (1) and press the box firmly down. The adapter (1) is now securely affixed to the surface. For additional safety, turn the box off the base adapter (1) to detach and firmly press down the edges of the adapter with your flat hand.

The base adapter (1) is now firmly secured to the surface. Empty boxes can be attached to the base adapter as often as required without the risk of overturning. When the box is full, lock the final closure and detach the box from the adapter by turning it counter-clockwise.

Note:

Turn the box clockwise for attachment and counter-clockwise to detach.

2. Wall holder with base adapter (REF 95.963)

Refer to Item 1 above: Base adapter.

3. Wall holder twin plus (REF 95.963.007)

Hang box into holder from above.

4. Wall holder euroMatic® (REF 95.963.001)

Hang box into holder from above.

5. Wall holder vario (REF 95.963.020)

Hang box into holder from above.

Accessories:

3.1 (4.1/5.1) Round bar holder for wall holder euroMatic®/vario/ twin plus (REF 95.963.006)

Compatible with the wall holder for round bars with a maximum diameter of 50 mm.

3.2 (4.2/5.2) Mounting panel for wall holder euroMatic®/vario/ twin plus (REF 95.963.005)

Compatible with the wall holder for system rails (e.g. Dräger) with a maximum thickness of 10 mm.

6. Recess in tray

Place the safety box into the moulded recess in the tray for a secure fit in the tray cavity.