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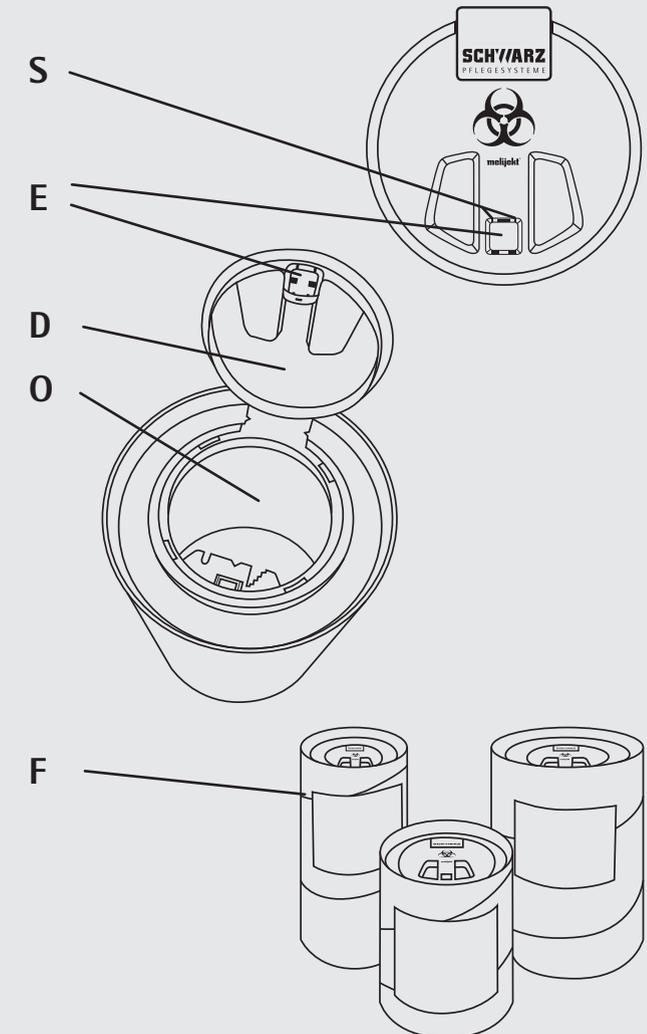
1. Intended use

The disposal drum is used for disposal of pointed and sharp objects such as syringes and scalpel blades from the medical field.

2. Safety instructions

- Use the appropriate drum size for the corresponding disposal item.
- Drum is intended for use in non-public areas only.
- Keep the drum out of the reach of children.
- Only drop objects vertically into the drum.
- Scalpels must be disposed of with care in the drum. Excessive force when inserting or filling with other objects may jam or damage the walls or floor.
- Do not push objects forcibly into the drum.
- Do not use the lid (D) to push in the objects to be disposed of.
- Do not pour any liquids into the drum.
- Do not fill the drum beyond the Fill Line (F) (danger of injury!).
- Do not put your hand in the drum (risk of injury!).
- Do not throw down the drum. Do not shake the drum. Do not drop the drum.
- Before closing the drum, make sure that no objects are protruding from the opening.
- Only close the drum with your hands and not forcefully.
- Do not squeeze the filled drum from the outside.
- Before disposing of the drum, make sure that the final closure (E) has been actuated.

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3. Daily use

For daily use, press the lid (D) into the filling opening (O) without activating the final closure pushbutton (E).

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After the drum has been filled (Caution: Do not fill above the filling line (F)) as follows: Place the drum on a solid surface and press the lid (D) completely into the filling opening (O) until a double snap is heard. Then push the pushbutton (E) marked with "PRESS FOR FINAL CLOSURE" firmly down with your thumb until it snaps in place (S) and points diagonally downwards. Check the final closure of the lid!

5. Fill Line – Maximum Fill Height (F)

In the upper part of the drum, a Fill Line (F) is visible. Never fill the drum beyond this Fill Line (F). The maximum Fill Height (F) must not be exceeded for safety reasons.

6. Disposal and labeling


The disposal drum consist of environmentally neutral paper and polypropylene. During combustion, no harmful substances are produced. When disposing, observe the legal requirements!

For use according to TRBA 250 and DIN EN ISO 23907 please use the optionally available absorbent insert.

Available sizes:

No.	Article	Item
01493511	melijekt®	Needle Disposal Drum 4 l
01494011	melijekt®	Needle Disposal Drum 6 l
01494511	melijekt®	Needle Disposal Drum 10 l
01494111	melijekt®	Needle Disposal Drum 11 l

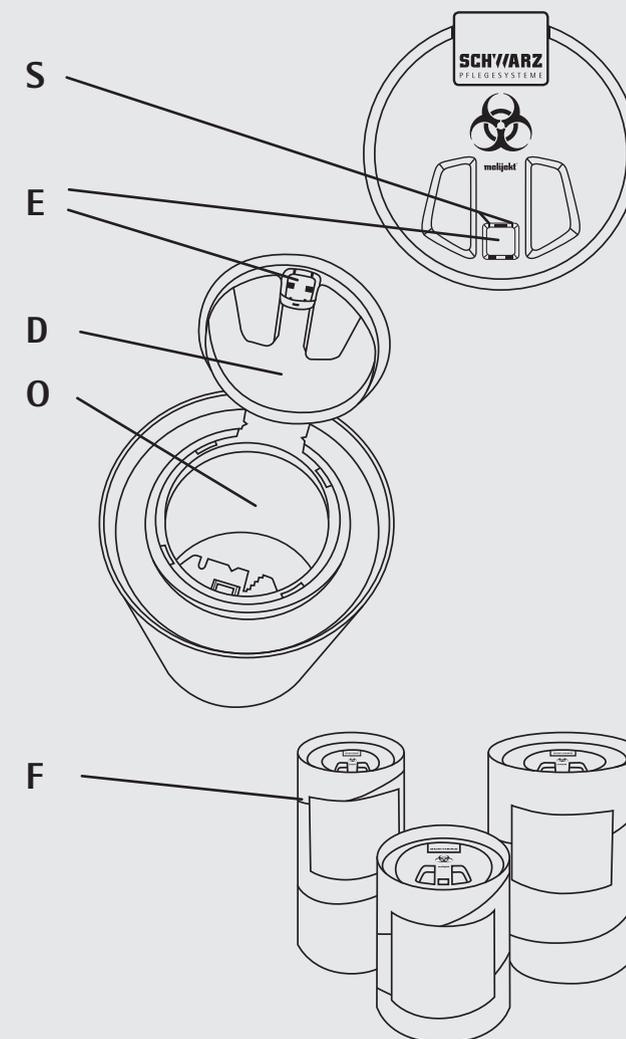Needle Disposal Drum Instructions for Use

12/2018 – subject to change

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Helmut Schwarz GmbH